

Gebauer's
Pain Ease[®]
 The **INSTANT** topical anesthetic.

**Bring instant relief to your patient's face
 IN 4 TO 10 SECONDS FLAT.**

Patient satisfaction and comfort is important to healthcare professionals. By using Pain Ease healthcare professionals can ease patient pain and anxiety. Pain Ease temporarily controls the pain of needle and minor surgical procedures.

Needle Procedures:

- IV Starts
- Blood Draws
- Immunizations
- Venipuncture
- Joint Injections
- Pre-Lidocaine Injections

Minor Surgical Procedures:

- I&D of Small Abscesses
- Lancing Boils
- Foreign Body Removal
- Wound Debridement
- Suturing
- Suture Removal
- Stapling

Target Markets:

- Hospital Systems and Hospital Based Practices
- Emergency Departments
- Urgent Care Centers
- Free Standing ER's
- Outpatient Surgery Centers
- Pediatric Practices

Pain Ease Advantage:

- Use on intact skin, minor open wounds and intact oral mucous membranes
- May be used by any licensed healthcare practitioner without the order of a physician
- Nondrug
- Nonflammable

Medium Stream Spray
 0386-0008-03
Mist Spray
 0386-0008-02
Single Patient Medium Stream Spray
 0386-0008-04



Determining the Ideal Spray Type for Your Customer

Medium Stream Spray

- Medium Stream is our most popular spray
- Produces a pinpoint stream
- Anesthetizes the skin at a specific location
- Ideal for procedures such as:
 - IV starts, blood draws, injections, incision and drainage of small abscesses, suturing and suture removal

Mist Spray

- Mist may be appropriate for broader application sites
- Causes a more intense cooling effect than Medium Stream spray due to faster evaporation
- Mist is more appropriate for facial applications



Important Risk and Safety Information for Gebauer's Pain Ease:

Published clinical trial results support the use in children three years of age and older. Do not use on large areas of damaged skin, puncture wounds, animal bites or serious wounds. Do not spray in eyes. Over spraying may cause frostbite. Freezing may alter skin pigmentation. Use caution when using product on diabetics or persons with poor circulation. Apply only to intact oral mucous membranes. Do not use on genital mucous membranes. The thawing process may be painful and freezing may lower resistance to infection and delay healing. If skin irritation develops, discontinue use. CAUTION: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.



4444 East 153rd. Street • Cleveland, Ohio 44128 | 1.800.321.9348 | www.Gebauer.com