

Make great removables with Myerson teeth & Myerson's advanced thermoplastic materials



With 4 strap strengths and 9 strap lengths, a custom EMA treats snoring and obstructive sleep apnea safely and effectively, giving you the sleep you need to lead a full, healthy life.



ema is a trademark of Franz design inc. US Patent Numbers: 6,105,265; D011,153 & Patent Pending

**DURAFLEX™**  
PARTIALS



**VISICLEAR®**  
CLASPS & FRAMEWORKS



**DURACETAL®**  
COSMETIC APPLIANCES



**myerson.**

Preformed plastic denture teeth for use in making temporary or removable dentures.

DISTRIBUTED BY  
**Myerson LLC**  
5106 North Ravenswood, Chicago, IL 60640-2713, United States  
toll free phone 800 423 2683, phone 312 432 8200, fax 800 424 2928  
[myersontooth.com](http://myersontooth.com)

**EC REP** EUROPEAN REPRESENTATIVE  
**Obelis s.a**  
Bd.Général Wahis 53  
1030 Brussel, Belgium  
phone +(32) 2.732.59.54  
fax +(32) 2.732.60.03  
email: mail@obelis.net

**MANUFACTURED BY**  
**Myerson Company Limited**  
3 Trinity Avenue  
Laventille  
Trinidad & Tobago  
phone 868 623 1007

Documented as per ISO 22112:2017  
Revision 2020-01-21

# Myerson Special Mould Chart

**myerson.**

We make the teeth that make the smile.



## myerson special... bring only the best

Myerson provides a premium denture solution with Myerson Special® teeth that meet the profession's highest standard in denture aesthetics. Myerson Special teeth come in an extensive selection of personalized moulds and all 16 A-D shades.

### Myerson Special Aesthetic Advantages

- Natural fluorescence: consistent shade values under any light conditions.
- Natural translucency & surface luster: harmonious with adjacent natural teeth.
- Natural labial carvings: perfectly suited for partial dentures

Characterizing features are individually added by hand to simulate the appearance of natural dentition. Colors and special effects come from the tooth rather than on the surface, which means that the teeth can be modified without affecting the shade. The result is a look that will satisfy even the most discerning patient.

**myerson.**

We make the teeth that make the smile.

myerson teeth mirror the beauty & strength of natural teeth.

Approximal stains Strong surface relief Enamel striation Darker neck of tooth



Myerson Special Anterior teeth are made with Myerson's time-proven "Durablend" formula, dentistry's original highly cross-linked copolymer resin, providing excellent resistance to wear and fracture.

Myerson Special anterior teeth are virtually indistinguishable from natural teeth, thanks to their "enamel-like" - opalescence and translucence combined, just as nature does in human enamel.

Myerson Special anterior teeth combine with Myerson DB Plus composite posterior teeth, in which our Durablend resin is reinforced with sub-micron silica, further enhancing their hardness and resistance to wear.

Composite filling Decalcification Translucent incisal areas Darker canine teeth.

## COMBINATION CHART

MAXILLARY ANTERIOR MOULD	ARTICULATES WITH MANDIBULAR MOULD	COMBINES WITH DB PLUS 0 POSTERIOR MOULD	COMBINES WITH DB PLUS F-10 POSTERIOR MOULD	COMBINES WITH DB PLUS 20 POSTERIOR MOULD	COMBINES WITH DB PLUS 30 POSTERIOR MOULD
A2	11	29	F29	29M or 29L	S30, L30 or 31
A4	7	29	F29	29M or 29L	S30, L30 or 31
A7	7	31	F31	31M or 31L	S32, L32 or 33
A9	8	31	F31	31M or 31L	S32, L32 or 33
A11	14	33	F33	33M	33 or X34
A13	16	33	F33	33M	33 or X34
A43	12	29	F29	29M or 29L	S30, L30 or 31
A44	13	31	F31	31M or 31L	S32, L32 or 33
A52	13	33	F33	33M	33 or X34
A53	16	33	F33	33M	33 or X34
A64	A71	33	F33	33M	33 or X34
A72	A73	33	F33	33M	33 or X34
YM3	11	29	F29	29M or 29L	S30, L30 or 31
YM5	8	31	F31	31M or 31L	S32, L32 or 33
YW3	13	31	F31	31M or 31L	S32, L32 or 33
YW5	14	31	F31	31M or 31L	S32, L32 or 33
PM2	11	29	F29	29M or 29L	S30, L30 or 31
PM4	12	31	F31	31M or 31L	S32, L32 or 33
PM6	13	31	F31	31M or 31L	S32, L32 or 33
PM8	14	33	F33	33M	33 or X34
PW2	13	31	F31	31M or 31L	S32, L32 or 33
PW4	A73	33	F33	33M	33 or X34
PW6	16	33	F33	33M	33 or X34
PW8	16	33	F33	33M	33 or X34




# Myerson SPECIAL MAXILLARY ANTERIOR TEETH



<b>A2</b>		40.3 9.5 7.3 (11)
<b>A4</b>		41.4 9.6 7.6 (7)
<b>A7</b>		43.5 9.9 7.9 (7)
<b>A9</b>		44.5 10.5 8.0 (8)
<b>A11</b>		48.0 11.1 8.8 (14)
<b>A13</b>		50.3 12.9 9.3 (16)
<b>A43</b>		43.2 9.2 7.9 (12)
<b>A44</b>		45.0 9.9 8.3 (13)
<b>A52</b>		45.8 10.4 8.3 (13)
<b>A53</b>		47.0 11.4 8.6 (16)

<b>A64</b>		47.0 9.7 8.6 (A71)
<b>A72</b>		49.0 10.4 9.1 (A73)
<b>YM3</b>		41.5 9.4 7.9 (11)
<b>YM5</b>		43.5 10.4 8.2 (8)
<b>YW3</b>		45.5 10.2 8.6 (13)
<b>YW5</b>		46.8 11.2 8.9 (14)

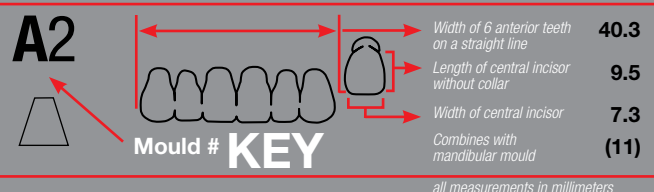

**Mould Variety**  
Myerson Special teeth are available in a wide variety of types and forms that make it possible to customize dentures to conform to the personal characteristics and facial features of most patients.

**Maxillary Anterior Moulds**  
Myerson anterior teeth are categorized into three mould types.

- A or Aesthetic Moulds**
  - Slender forms with excellent balance between ridge lap & bite
  - Subtle labial carving and lifelike blending
- Y or Youthful Moulds**
  - These are similar to A moulds in aesthetic concept except that they have a shorter ridge lap
  - Beautiful sculpting with a vigorous youthful look
  - Longer more youthful lingual line
  - Fewer striations and age related effects than P moulds
- P or Prime Moulds**
  - Designed for older patients
  - Shorter, less sloping ridge laps to help butting the ridge in partial and immediate dentures
  - Excellent carvings with more pronounced age characteristics, ruggedness and curvature

*all measurements in millimeters*

# MAXILLARY

<b>PM2</b>		41.8 9.0 7.8 (11)
<b>PM4</b>		43.8 10.0 8.1 (12)
<b>PM6</b>		45.8 11.0 8.4 (13)
<b>PM8</b>		47.2 12.0 8.7 (14)
<b>PW2</b>		46.4 9.8 8.5 (13)
<b>PW4</b>		47.8 10.8 8.8 (A73)
<b>PW6</b>		48.5 11.8 9.1 (16)
<b>PW8</b>		50.0 12.8 9.2 (16)

<b>8</b>		33.1 9.8 5.0
<b>10</b>		34.3 10.1 5.2
<b>11</b>		31.0 8.5 4.4
<b>12</b>		32.9 8.5 4.6
<b>13</b>		34.7 9.8 5.1
<b>14</b>		34.9 10.8 5.1
<b>16</b>		36.7 11.1 5.6
<b>17</b>		36.8 12.3 5.6
<b>A63</b>		34.7 8.0 5.4
<b>A71</b>		35.0 9.0 5.2
<b>A73</b>		36.5 9.0 5.5

# MANDIBULAR

<b>3</b>		29.2 9.0 4.4
<b>7</b>		32.0 9.1 4.6

# myerson DB PLUS 30 DEGREE POSTERIOR TEETH

<b>31</b>		31.3 33.9 9.50 10.15
<b>33</b>		31.8 35.7 9.60 9.82
<b>S30</b>		28.5 29.4 9.12 8.53
<b>S32</b>		31.6 32.0 9.54 9.09

<b>L30</b>		28.8 30.2 9.10 9.15
<b>L32</b>		31.2 31.1 9.15 9.15
<b>X34</b>		33.0 35.8 9.94 9.74

# BICUSPIDS ONLY

<b>X36</b>		15.7 Width of 2 9.50 10.15
------------	--	----------------------------------

# myerson LINGUALIZED INTEGRATION

<b>MC</b>		31.2 32.3 9.30 8.50
-----------	--	------------------------


<b>CC</b>		30.9 32.3 9.10 8.50
-----------	--	------------------------

# myerson DB PLUS 0 DEGREE

<b>29</b>		29.0 30.5 7.10 7.85
<b>31</b>		30.5 32.3 7.55 8.50
<b>33</b>		32.8 34.9 8.02 8.74
<b>F-10</b>		
<b>F29</b>		28.6 31.6 8.36 7.48
<b>F31</b>		31.0 31.8 9.19 8.37
<b>F33</b>		32.8 34.3 9.62 9.09
<b>F35</b>		36.5 37.5 10.8 9.62

# 20 DEGREE

<b>29M</b>		29.0 31.6 8.10 8.85
<b>29L</b>		29.0 31.4 8.85 8.70
<b>31M</b>		31.4 31.6 8.10 9.10
<b>31L</b>		31.8 34.8 9.55 9.25
<b>33M</b>		33.1 35.8 8.35 9.40

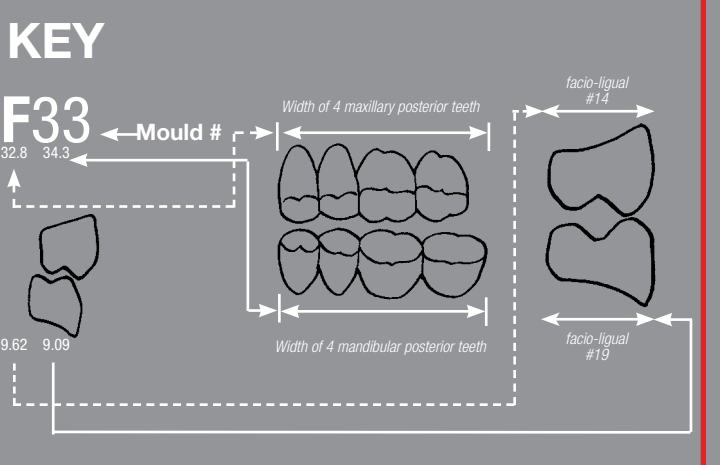

**myerson**  
We make the teeth  
that make the smile.

With 95 years of service to the denture profession, myerson can help you make every denture case a successful one. Use myerson made teeth on your next complete or partial denture case and you'll join the growing number of dental professionals who have placed their trust in myerson.